



LumiraDx™ SARS-CoV-2 Ag Ultra Test Technical Bulletin – Swabs

The LumiraDx SARS-CoV-2 Ag Ultra test is an automated rapid microfluidic immunofluorescence assay for use with the LumiraDx Platform, for near-patient testing, intended for the qualitative detection of the nucleocapsid protein antigen from SARS-CoV-2 directly from anterior nasal swab samples collected from individuals suspected of COVID-19 by their healthcare provider within the first twelve days of symptom onset or from individuals without symptoms or other epidemiological reasons to suspect COVID-19.

This Technical Bulletin will aim to provide a comprehensive list of LumiraDx validated swabs for use with the SARS-CoV-2 Ag Ultra test.

SARS-CoV-2 Ag Ultra test – Approved Swabs

- Copan Nasal FLOQSwab™ Regular with 80mm breakpoint (Supplier Product Code: 502CS01)
- Copan Nasal FLOQSwab™ Regular with 100mm breakpoint (Supplier Product Code: 519CS01)
- Copan Nasal FLOQSwab™ Regular with 30mm breakpoint (Supplier Product Code: 520CS01)
- Copan Nasal FLOQSwab™ Regular with no breakpoint (Supplier Product Code: 5U060S01)
- Copan FLOQswab Regular Tip (Supplier product code: 519C)
- Copan FLOQswab Regular Tip (Supplier product code: 552C)
- Copan FLOQswab Regular Tip (Supplier product code: 520C)
- Copan FLOQSwab mini- FLEXIBLE with 100mm Breakpoint (Supplier Product Code: 516CS01)
- Copan FLOQSwab mini-tip with 100mm Breakpoint (Supplier Product Code: 518CS01)
- Copan FLOQSwab mini-tip with 80mm Breakpoint (Supplier Product Code: 501CS01)
- Puritan HydraFlock™ Sterile Standard Flock Swab (Supplier Product Code: 25-3506-H)
- mwe medical wire DrySwab™ Rayon Swab (Supplier Product Code: MW112)
- Medico Technology Co., Ltd Specimen Collection Swab (Supplier Product Code MFS-94000KQ)
- Gongdong Disposable Sterile Swab (Supplier Product Code: G1029FT77)

This Technical Bulletin includes a complete list of swabs specific to a supplier and a product code whose performance was confirmed acceptable for use with the LumiraDx SARS-CoV-2 Ag Ultra test at the time of validation testing. LumiraDx cannot guarantee that swab suppliers will not change raw materials or other swab specifications, which could affect performance.

Swabs from alternative suppliers have not been validated for use with the LumiraDx SARS-CoV-2 Ag Ultra test. The Test may not perform as expected if using swabs from alternative suppliers.

Furthermore, the above suppliers may offer similar swabs under a different product code. These swabs have not been validated for use with the LumiraDx SARS-CoV-2 Ag Ultra test. The test may not perform as expected if using swabs other than the exact product codes listed above.

For product enquiries and technical support please contact LumiraDx Customer Services.

E-mail: customerservices@lumiradx.com

Telephone: +44 (0)1172841222

www.lumiradx.com

Copyright © 2022 LumiraDx UK LTD. All rights reserved worldwide.
LumiraDx and Flame logo are trademarks of LumiraDx International LTD. Full details of these and other registrations of LumiraDx can be found at lumiradx.com/IP. All other trademarks are the property of their respective owners.
Content should be used for the use of the LumiraDx products only and in line with instructions provided. You may not, except with our express written permission, distribute or commercially exploit the content. Nor may you transmit it or store it in any other form of electronic retrieval system other than for the purpose of use of the LumiraDx Instrument or LumiraDx Test Strips. Information provided is subject to change without notice.

Product is not available in all countries and regions. Please check with your local LumiraDx sales representative or distributor for availability in specific markets.

Manufactured by:
LumiraDx UK Ltd
Dumyat Business Park
Alloa
FK10 2PB, UK

Registration Number:
09206123

**Authorized representative
in the European Union:**
LumiraDx AB,
Västra Vägen 5A,
16961 Solna, Sweden

S-COM-ART-02410 R1