lumiraDx® D-Dimer Quality Controls

For Professional Use Only
SPEC 31067 R2
ART-00238 R3

The LumiraDx D-Dimer Quality Controls (hereafter referred to as Quality Controls) are optional liquid quality controls to be used with the LumiraDx Instrument (hereafter referred to as the Instrument) and the LumiraDx D-Dimer Test (hereafter referred to as D-Dimer Test).

Read these instructions thoroughly before using the Quality Controls.

Inspect the Quality Control packaging and contents for damage before use. Report any damage to LumiraDx Customer Services and do not use the kit if any damage is reported to the contents.

To ensure that you are using the Instrument, the D-Dimer Test and the Quality Controls correctly, read the appropriate Platform User Manual, D-Dimer Test Product Insert and this entire pack insert. In addition, please watch the LumiraDx Platform Training video available at kc.lumiradx.com. The Quality Controls are intended for professional use only. The LumiraDx Logo and LumiraDx are trademarks of the LumiraDx Group.

Intended use

The LumiraDx D-Dimer Quality Controls are intended for liquid quality control testing performed on the LumiraDx Instrument when used with the LumiraDx Test Strip. The Quality Controls provide users with assurance that the device is performing within specification.

Summary and explanation of the test

The LumiraDx D-Dimer Quality Controls are an optional quality control for the Instrument when used with the LumiraDx D-Dimer Test. The Quality Control material is a device intended for medical purposes for use in a test system to estimate test precision and to detect systematic analytical deviations that may arise from reagent or analytical instrument variation and may be used for proficiency testing.

Warnings and precautions

- For *in vitro* diagnostic use.
- This control contains human source material that was tested and found nonreactive for the Human Immunodeficiency Virus (HIV 1 and 2) antibody, Hepatitis B Surface Antigen (HbsAg) and Hepatitis C Virus (Anti-HCV) at the donor stage. This product, as with all human based specimens, should be treated as potentially infectious and handled with proper laboratory safety procedures to minimize the risk of transmission of infectious disease.
- All components of this kit can be discarded as Biohazard waste according to the local guidelines.
- Refer to the product safety data sheet for risk and safety phrases and disposal information.
- The product safety data sheet is available for users upon request.
- Requirements of the appropriate licensing or accrediting body should be incorporated into your quality control program.
- Exercise the normal precautions required for handling all laboratory reagents.

Storage and stability

- Store controls between 2°C and 8°C (36 - 46°F). **DO NOT FREEZE.**
- Opened controls are stable for 45 days between 2°C and 8°C (36 and 46°F) if kept capped in original container and free from contamination.
- Unopened controls that are stored between 2°C and 8°C (36 and 46°F) can be used until the expiration date.

Carton contents

3 or 6 x 3ml vials Level 1 Quality Control

3 or 6 x 3ml vials Level 2 Quality Control

D-Dimer Quality Controls Pack Insert

Materials required but not provided with the control carton

- LumiraDx Instrument
- LumiraDx D-Dimer Test Strips
- LumiraDx Connect- if connectivity required (refer to LumiraDx Connect User Manual)
- Quality Control Ranges Product Insert as included in the D-Dimer Test Strip Carton

Getting ready to test

You will need the LumiraDx Instrument and the following supplies:

- LumiraDx D-Dimer Test Strip(s)
- LumiraDx D-Dimer Quality Controls Level 1 or Level 2
- Transfer pipette

Preparing the Quality Controls

The liquid controls are supplied ready to use.
Handling the LumiraDx D-Dimer Test Strips

To ensure that you are using the D-Dimer Test and the Instrument correctly, read the appropriate D-Dimer Test Strip Product Insert and Platform User Manual.

Procedure/performing a test

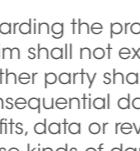
Consult the LumiraDx Platform User Manual for instructions on how to analyse a Quality Control sample. Open the foil pouch of the D-Dimer Test Strip just before use and insert the Test Strip into the LumiraDx Instrument. The Instrument will indicate when ready for the sample to be applied.

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
Manipulation des Bandelettes-test D-Dimer LumiraDx

1. Draw up the **Quality Control solution** into the transfer pipette.

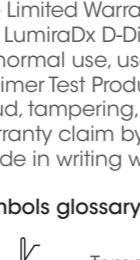2. Apply the **Quality Control solution** to the **already inserted D-Dimer Strip**. Hold the Strip over the Sample Application Area of the Test Strip and dispense the Quality Control solution. The LumiraDx Instrument will indicate sample is detected with an audible alert (if the Instrument sounds are enabled). The screen of the LumiraDx Instrument will request the user to close the door. Dispose of pipette.

3. **Do not open the door while the test is in progress.** The touch-screen will indicate test progress.

4. The **result** will appear on the touch-screen within 5 - 7 minutes of applying the Quality Control solution and starting the test.

5. NOTE: If you need to repeat a test, use a new Test Strip.

Expected results

The Instrument displays the result acceptable range and Pass or Fail. The Quality Control Ranges Product Insert in the D-Dimer Test Strip carton may also be consulted to confirm the result is within the acceptable range. The result is automatically saved in the memory of the Instrument. The system is working properly and all handling has been done correctly when the test results obtained are within the acceptable control range as written in the Quality Control Ranges Product Insert.

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Performance characteristics

Quality Control precision was determined in measurement system analysis for the Quality Controls with the LumiraDx D-Dimer Test. The results were generated over multiple days, by multiple operators and instruments. The results are shown in µg/L FEU (Fibrinogen Equivalent Units).

Manufacturer information

LumiraDx UK Ltd 09206123, Dumyat Business Park, Alloa FK10 2PB, UK Registration number: 09206123

Quality Control Level 1

Mean	SD	%CV	N
918	98.98	9.13	1050

Quality Control Level 2

Mean	SD	%CV	N
2190	300.65	10.37	1050

Limitations

This product is designed as a liquid control for quality control performed exclusively by the LumiraDx D-Dimer Platform. The quality control result is subjected to the limitations of the test Platform. Deviations may indicate potential problems with one or more components in the test Platform. The LumiraDx Instrument and D-Dimer Test Strips have on-board controls to detect errors and prevent false D-Dimer results when analysis is performed. Therefore deviations observed when testing with the LumiraDx D-Dimer Quality Controls would not invalidate previous results obtained from LumiraDx D-Dimer tests.

LumiraDx Customer Service

If you have questions contact **LumiraDx Customer Services**, contact details available at lumiradx.com.

Any adverse results experienced with the use of this product, and/or quality problems should also be reported to LumiraDx Customer Services at the number available via the website lumiradx.com or by email: customerservices@lumiradx.com.

Getting ready to test

You will need the LumiraDx Instrument and the following supplies:

- LumiraDx D-Dimer Test Strip(s)
- LumiraDx D-Dimer Quality Controls Level 1 or Level 2
- Transfer pipette

Preparing the Quality Controls

The liquid controls are supplied ready to use.
Handling the LumiraDx D-Dimer Test Strips

To ensure that you are using the D-Dimer Test and the Instrument correctly, read the appropriate D-Dimer Test Strip Product Insert and Platform User Manual.

Procedure/performing a test

Consult the LumiraDx Platform User Manual for instructions on how to analyse a Quality Control sample. Open the foil pouch of the D-Dimer Test Strip just before use and insert the Test Strip into the LumiraDx Instrument. The Instrument will indicate when ready for the sample to be applied.

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