# lumiraDX™ HbA1c Quality Controls

For Professional Use Only

REF L006080101003

SPEC-35868 R1  
ART-02674 R1 Date of Rev 2022-08

The Lumira HbA1c Quality Controls (hereafter referred to as Quality Controls) are optional quality controls to be used with the LumiraDX Instrument (hereafter referred to as the Instrument) and the LumiraDX HbA1c Test Strip Product Insert and Platform User Manual.

Read the Quality Control packaging and contents for storage before use. Report any damage to LumiraDX Customer Services and do not use if any damage is observed to the contents.

The Quality Controls are intended for professional use only.

**Intended Use:**  
The Lumira HbA1c Quality Controls are intended for use by laboratory professionals/heathcare professionals for automated quality control testing performed on the LumiraDX Instrument when used with the LumiraDX HbA1c Test Strip. The Quality Controls provide users with assurance that the device is performing within specification.

To ensure that you are using the Instrument, the specific assay test and the Quality Controls correctly, read the appropriate Platform User Manual, the specific assay Test Product Insert and this Product Insert. In addition, please watch the LumiraDX Platform training video available at lumiradx.com. The Quality Controls are intended for professional use only.

**Summary and explanation of the test:**

The Lumira HbA1c Quality Controls are an optional quality control for the Instrument when used with the LumiraDX HbA1c Test Strip. The Quality Controls provide users with assurance that the device is performing within specification to estimate test precision and to detect systematic analytical deviations that may arise from reagent or analytical instrument variation and may be used for proficiency testing. Quality Control testing is at the discretion of your organization and the frequency of testing will be determined by local guidelines.

**Requirements:**  
Each Quality Control contains human whole blood stabilizers, a specified level of glycated hemoglobin (HbA1c), three Quality Control reagents supplied by the LumiraDX HbA1c Test Strip. The LumiraDX HbA1c Test Strip performance is traceable to the International Federation of Clinical Chemistry and Laboratory Medicine (IFCC) primary reference method for the measurement of HbA1c.

**Warnings and precautions:**  
For in vitro diagnostic use.

This control contains human source material that was tested and found non-reactive for the Human Immunodeficiency Virus (HIV 1 and 2), hepatitis B & Surface Antigen (HBsAg) and Hepatitis C Virus (Anti-HCV) at the donor stage. This product, as with all human based specimens, should be treated as infectious until the result of the test is known.

The Control contains 1% antimicrobial agents and stabilizers. Avoid ingestion or contact with skin or mucous membranes. In case of skin contact, wash affected area with copious amounts of water. Seek medical attention.

• Exercise the normal precautions required for handling all laboratory reagents.

• All components of this can be discarded as biohazardous waste according to the local guidelines.

• Refer to the product safety data sheet for risk and safety phrases and disposal information. The product safety data sheet is available at lumiradx.com.

• Requirements of the appropriate licensing or operating body should be incorporated into your quality control program.

**Storage and stability:**

• Store controls between 2°C and 8°C (36-46°F). DO NOT FREEZE.

• Allow the Quality Controls material to come up to room temperature (15° - 30°C; 59° - 86°F) for at least 10 minutes (but not above two (2) hours) before use.

• Unopened controls that are stored between 2°C and 8°C (36-46°F) can be used until the expiration date.

• Controls are stable for 30 days between 2°C and 8°C (36-46°F) after opening.

**Materials provided:**

• 3 x 0.5 mL vials Level 1 Quality Control stabilized whole blood

• 3 x 0.5 mL vials Level 2 Quality Control stabilized whole blood

• 40 x 20μL Single-Bulb Plastic Transfer Pipettes (single use)

• LumiraDX HbA1c Quality Controls Pack Insert

**Materials required but not provided with the Quality Control carton:**

• LumiraDX Instrument

• LumiraDX HbA1c Test Strips

• LumiraDX Connect - if connectivity required (refer to LumiraDX Connect User Manual)

• Quality Control Ranges Product Insert (included in the HbA1c Test Strip Carton)

**Getting ready to test:**

You will need the LumiraDX Instrument and the following supplies:

• LumiraDX HbA1c Test Strip(s)

• LumiraDX HbA1c Quality Controls Level 1 or Level 2

• Single-Bulb Plastic Transfer Pipettes (single use)

**Preparing the Quality Controls:**

The liquid Quality Controls are supplied ready to use.

**Handling the LumiraDX HbA1c Test Strips:**

To ensure that you are using the HbA1c test and the Instrument correctly, read the appropriate HbA1c Test Strip Product Insert and Platform User Manual.

The liquid Quality Controls are supplied ready to use.

**Procedure / Performing a Test:**

Consult the LumiraDX Platform User Manual for instructions on how to analyse a Quality Control sample. Open the foil pouch of the HbA1c Test Strip just before use and insert the Test Strip into the LumiraDX instrument. The instrument will indicate when ready for the sample to be applied.

Do not use the HbA1c Test Strip if there are any visible signs of damage to the foil pouch. Do not use the Test Strip if there are any visible signs of damage to the foil pouch or transfer pipette (equivalent).

When transferring the Quality Control solution from the vial to the Test Strip or transfer pipette (equivalent), copiable of accidentally dispensing 20μL of Quality Control solution is used. The Quality Control solution is applied directly from the vial to the sample application area of the Test Strip. No lysis buffer step is required to use the Quality Control solution.

Please note that the following transfer pipettes are acceptable for use with the LumiraDX HbA1c Quality Control solution:

• 20μL Single-Bulb Plastic Transfer Pipettes (single use)

• Calibrated Laboratory Pipette (set to 20μL with appropriate tips)

Please note that any transfer pipettes used with the LumiraDX HbA1c Quality Controls should never contain any coating additives, surfactant, or preservatives.

A single 20μL of the Quality Control solution from the vial to one of the following transfer pipettes:

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Aspirer une量的 Quality Control solution from the vial to one of the following transfer pipettes:

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